DESTORNILLADOR MANUAL Y LLAVES ACCESORIAS GALIMPLANT
GALIMPLANT MANUAL SCREWDRIVER AND ACESORY WRENCHES
CHAVE DE PARAFUSOS MANUAL E CHAVES ACESSÓRIAS GALIMPLANT

Español

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English

IMPORTANT! Read the information contained in these operating instructions carefully before using the product.

NUEVA GALIMPLANT complies with all the regulations established by the European laws and directives relating to the manufacture and distribution of medical products. The galimplant® system, attachments and instruments are certified and authorised for sale by a Notified Body, guaranteeing the perfect quality of all its products. The use and application of the galimplant® system is not under the manufacturer's control, and the user is responsible for any damages which may occur due to an incorrect manipulation or use of the product.

Safety, responsibility and guarantee

We guarantee the quality of our products within the framework of our sales and supply conditions. This product is part of a global concept and should only be used in conjunction with the associated original products according to the Nueva Galimplant instructions and recommendations. The non-recommended use of products manufactured by third parties together with galimplant® products will void any guarantee or other obligation, express or implicit, by Nueva Galimplant, S.L.U.

The user has sole responsibility for the procedure and use of the product. It is the practitioner's responsibility to use the product in accordance with the current instructions for use and to determine if it is appropriate for each patient's individual situation. In particular, this responsibility is expressly applicable in the case of its use in non-recommended procedures. If in doubt, the user should consult us.

Assistance and training

Technical assistance concerning the use of our products will be carried out verbally, by written means, by electronic means or by demonstrations. Said assistance will be given according to our knowledge of the current state of science and techniques when the product is on sale. This does not exempt the user from their obligation to personally make sure that the product is appropriate for the purposes and procedures planned. Nueva Galimplant, with the aim of ensuring the correct use of its products, offers courses and practical sessions about it products through the platform: galimplant.training.

Precautions and warnings

Galimplant® manual screwdriver and accessory wrenches have not been sterilised before delivery, so they must be cleaned, disinfected and sterilised before use and it is essential that they are handled in an aseptic manner. The following descriptions are insufficient for the immediate application of galimplant® manual screwdriver and accessory wrenches. This system should only be used by dentists, doctors and surgeons. The dentist/doctor/surgeon must have knowledge in oral implantology and experience in the manipulation of the product in order to use them safely and correctly following the indications of these instructions for use.

Description

The galimplant® manual screwdriver is made of surgical steel and is provided at its end with a 4x4mm square tubular wrench that fits over the long implant transfer and over the galimplant® short and long prosthetic wrenches. In its handle, the screwdriver has a coupling that allows the adaptation of the torque wrench to control both the insertion torque of the implant and the torque of the multi-position abutments.

The galimplant® accessory wrenches for the screwdriver are the ones described below:

- Short prosthetic wrench: device made of 16mm long surgical steel consisting of a 1.31mm hex head and a 4x4mm square end that fits over the manual screwdriver for the placement of galimplant® multi-position angled abutments and controls the torque of the same.
- Long prosthetic wrench: device made of 25mm long surgical steel consisting of a 1.31mm hex head and a 4x4mm square end that fits over the manual screwdriver for the placement of galimplant® multi-position angled abutments and controls the torque of the same.
- Long implant transfer: device made of grade 5 titanium which fits on its 4x4mm square end on the manual screwdriver and at its other end it adapts directly to the implant, once the implant transfer supplied with this implant has been removed.

Indications

The galimplant® manual screwdriver is indicated for the placement of galimplant® conventional implants and galimplant® zygomatic implants and as a support for the short and long prosthetic wrenches, which allows the positioning and controlled tightening of the multi-position angled abutments.

In the cases of the presence of adjacent teeth, bone prominences or other implants, the long implant transfer connected to the manual screwdriver allows direct access to the implant from a more distant position, which facilitates its insertion.

Since the handle of the manual screwdriver has an increased diameter compared to conventional screwdrivers, the user exerts the action with the entire palm of his hand, so that said instrument transfers a high torque. That is why Nueva Galimplant recommends its controlled use with a torque wrench.

Patient Groups and Clinical Benefits

The galimplant® manual screwdriver with direct implant connection are intended for use in patients subject to dental implant treatment.

The clinical benefit of treatment is the replacement of the patient's missing teeth.

Contraindications

The relevant contraindications are those relating to surgical implant procedures in patients who are medically unfit for an oral surgical procedure or who are allergic or hypersensitive to stainless steel and or titanium grade 5.

Side Effects and complications

The secondary effects, interactions and complications are those which derive from the implant fitting proceedings.

Storage and transport

The galimplant® manual screwdriver and accessory wrenches are delivered packed in peel-pack bags inside a closed cardboard blister on which the label is placed and includes all product information, including the date of manufacture, which can be deduced from the batch number, the first four digits corresponding to the year and the next two to the month of manufacture (YYYYMMXXX). This packaging contains and protects the product during storage and transport does not have a specified shelf-life.

The manual screwdriver and accessory wrenches are made of with stable materials under normal environmental conditions, but it is recommended to store them in the original packaging and not expose them to extreme conditions (temperatures, humidity, shock, etc.) that may affect the packaging and labeling.

When removing the products from its packaging one must verify that it is not damaged and that the product fits the description on the label.

Cleaning, disinfection and sterilisation

The galimplant® manual screwdriver and accessory wrenches are reusable and are sold in a NON-STERILE state. In accordance with the EN ISO 17664 Normative, Nueva Galimplant will provide all of the information necessary to process their medical devices labelled as "Non sterile" and which need to be cleaned, disinfected and sterilized prior to use, both for the first use and for all subsequent usages. The galimplant® manual screwdriver and accessory wrenches are included in said medical devices.

Warnings:

- Do not reuse the products without prior cleaning, disinfection and sterilisation. The new "non-sterile" products should be processed in the same way as the used products.
- For their own safety, staff should use adequate protective clothing and equipment at all times: gloves, mask, protective glasses, etc;
- Nueva Galimplant recommends the use of aldehyde free cleaner and disinfectants with proven effectiveness, in order to avoid the fixation of blood proteins (included on the VAH/DGHM list or which have the CE Marking).
- The use of neutral or slightly alkaline cleaning agents is recommended by Nueva Galimplant.
- It is essential to verify that the cleaning product used is compatible with the medical devices are going to processed and respect the specifications of use (temperature, concentration and time) indicated by the manufacturer.
- Nueva Galimplant cannot control the clinical manipulation procedures, the cleaning methods or any other user level conditions, therefore they will not be held liable for the sterilization of the products even when the recommended guidelines contained in this instruction manual have been followed.

Limitations and restrictions:

The repetition of the indicated processes has a minimal effect on the galimplant® products. The useful life of these is normally determined by the wear and tear and damage caused due to repeated use.

The best way to assess the state of deterioration is by carrying out a thorough and functional check prior to use. Worn down cutting edges or signs of corrosion are indications that said products is not suitable for use and it should be disposed of.

Preparation in the place of use

As soon as possible following their use, the galimplant® products should be transferred to the place where they are to be cleaned, in order avoid the drying-on of any blood residues, tissues, secretions or bone fragments. If it is likely that the cleaning process will be delayed, these should be covered with a moist cloth or stored in an airtight container filled with water in order to delay the drying-on of the organic material and to avoid the risk of contamination. To avoid contamination, the contaminated products should not be put back into the surgical box. The used devices should be placed in a container that has been set up for the pre-cleaning which shall be carried out as per the following indications.

Preparation before cleaning

This stage must be carried out regardless as to whether the cleaning and disinfection processes are carried out manually or automatically.

- As soon as possible following their use, immerse the galimplant® products in a container that contains a cleansing solution with proteolytic enzymes, prepared in accordance with the manufacturer's specifications, respecting the temperature, concentration and immersion time. These enzyme solutions separate proteins and ensure that blood and other secretions do not dry on the devices.
- Make sure that all the product's surfaces are covered by the solution and that the instruments do not touch each other.
- Manually brush with a nylon bristle brush (never metal, wool or steel brushes) until all of the visible dirt and residue has been eliminated, paying special attention to the zones which may not be reached by the brushing action, such as grooves or internal parts.
- Remove the products from the container containing solution and rinse them thoroughly with plenty of water, preferably purified water, to ensure the reduction of organic material from mechanical drag.

Cleaning and disinfection

To clean and disinfect the products either of the two following methods may be used:

Manual Method

- Place the galimplant® products in a perforated tray or basket.
- Prepare an ultrasonic bath and a detergent and disinfectant solution in accordance with the manufacturer's specifications of the product used.
Nueva Galimplant has validated this process using the product neodisher® septo PreClean diluted at 1% to prepare the ultrasonic bath.
- Immerse the basket with the galimplant® products in the ultrasonic bath and keep them at a temperature of 20° C for 15 minutes.
- Remove the products from the ultrasonic bath and rinse with plenty of purified water.
- If necessary, repeat the cleaning and disinfection step.
- Dry the devices using filtered compressed air and single use lint free cloth.

Automatic Method

Place the galimplant® products in the thermal washer-disinfector's tray or basket.

Nueva Galimplant S.L.U. recommends thermal washer-disinfectors that comply with the EN ISO 15883 normative.

- Run the cleaning, disinfecting and drying cycle.
Nueva Galimplant has validated the Vario-TD program by the professional brand Miele, using the detergent neodisher® MediClean forte diluted at 0.5%. This program consists of:
 - Cold water soak for 4 minutes.
 - 55° C clean for 10 minutes with detergent.
 - Cold water neutralisation for 6 minutes.
 - Cold water mid-cycle rinse for 3 minutes.
 - 93° C thermal disinfection for 5 minutes.
 - 110° C dry for 25 minutes.

To guarantee optimum protection of the products we recommend that purified water is used throughout the whole process.

Note: it is important to always follow the instructions provided by the detergent and thermal washer-disinfector manufacturers.

Inspection, maintenance and testing

- Carefully inspect all of the galimplant® products in order to check if any damage has been caused to them (corrosion, loss of shape, breaking) or contamination. We recommend that the inspection is carried out in a well-lit environment or using a magnifying lens in order to improve visibility.
- If there are any traces of contamination on any of the devices the cleaning and disinfection process should be repeated.
- If there are any damage (deformation, failure, ...), the product should be discarded.

Packaging

Once they are clean, disinfected and completely dry, the products must be packaged in a surgical box and then the box should be packaged in a sterilisation bag.

The packaging of thermally sterilised medical devices must comply with the following requisites:

- Be appropriate for steam sterilisation, that is, have a minimum thermic resistance of 137° C and be sufficiently permeable for steam.
- Protect the individual devices or surgical boxes sufficiently from mechanical damage.
- Comply with the EN ISO 11607 normative.

Once the devices have been packaged a label should be added to each sterilisation bag which contains data such as the sterilisation date and it should include the reference, batch and the number of devices if the bag does not have a transparent window which allows the user to see what is inside.

Sterilisation

- Load the autoclave with the conditioned galimplant® products in the sterilisation bags. Do not overload the autoclave and always follow the equipment manufacturer's instructions.
- Just like in the other steps of these instructions, we recommend the use of purified water to guarantee optimum quality.
- Nueva Galimplant recommends using an autoclave which complies with the requisites of the EN ISO 17665 normative, in addition to being validated and having adequate maintenance.
The validation of the sterilisation was carried out in a steam autoclave using a 134° C pre-vacuum cycle for 6 minutes.
- To control the sterilisation process, the use of chemical indicators is recommended, such as, for example, the verification strips which the sterilisation bags come equipped with and which change colour when the sterilisation process has finished.

Note: it is important to always follow the autoclave manufacturer's instructions and keep maintenance up to date.

WARNINGS

Use the manual screwdriver and accessory wrenches only for the insertion of original galimplant® dental implants.

The practitioner must provide the patient with the information about side effects, interactions and complications in relation to galimplant® products.

The clinical practitioner must verify that the materials and techniques to be used during surgical procedure are appropriate and compatible with the galimplant® manual screwdriver and accessory wrenches and must follow the manufacturer's instructions for use of these materials.

Failure to observe the procedures set out in these instructions could harm the patient and give rise to one or all of the following complications:

- Aspiration or swallowing of the components.
- Infection.
- Damage to the implant or instruments.
- Implant failure.

In the event of a serious incident related to the use or result of use of the product, inform Nueva Galimplant (www.galimplant.com) and your National Competent Authority.

Procedures for the disposal of the product

If used, dispose the manual screwdriver and accessory wrenches as biological waste, according to the local regulations.

Additional information

For more information concerning the use of galimplant® products, contact your local representative or visit our website www.galimplant.com

Validity

These instructions for use will be valid as of May 2021.

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